



Raised Toilet Seat



B3125

USER MANUAL - RAISED TOILET SEAT

GENERAL EXPLANATION OF YOUR RAISED TOILET SEAT



USER GUIDE

⚠ Prior to use please read all instructions.

We recommend you keep this User Guide in a safe place for future reference.

TABLE OF CONTENTS

SPECIFICATIONS	2	· Intended for individuals who have difficulty sitting down or getting up from the toilet
SET UP	3	· All composite design prevents germ build up and cleans easily
CLEANING	3	· Fits most standard and elongated commodes
WARRANTY	3	· Adds 4" to height of toilet seat · Lightweight and portable · Simple, tool-free installation

SPECIFICATIONS

NAME	Raised Toilet Seat
WEIGHT CAP	300 lbs

SET UP

Seat Assembly

- 1 Lift toilet seat and toilet lid [A].
- 2 Place Raised Toilet Seat on brim of toilet [B].
- 3 Ensure that the Raised Toilet Seat is level and stable before use [C].



CLEANING

! Do not use high pressure cleaners, steam cleaners or harsh, caustic chemicals for cleaning.

- › Use a non-abrasive detergent or household cleaner with warm water.
- › Rinse with warm water and wipe dry.
- › DO NOT USE AN ABRASIVE CLEANER OR CLOTH with this product, as damage may ensue.

LIFETIME LIMITED WARRANTY

Your Lifestyle Mobility Aids product is warranted to be free of defects in materials and workmanship for the lifetime of the original consumer purchaser.

This device was built to exacting standards and carefully inspected prior to shipment.

This Lifetime Limited Warranty is an expression of our confidence in the materials and workmanship of our products and our assurance to the consumer of years of dependable service.

In the event of a defect covered by this warranty, we will, at our option, repair or replace the device.

This warranty does not cover device failure due to owner misuse or negligence, or normal wear and tear.

The warranty does not extend to non-durable components, such as rubber accessories, casters, and grips, which are subject to normal wear and need periodic replacement.



Some Additional Products:



Premium Shower
Chair with Back
and Padded Arms



Commando Memory
Forearm Crutch



Super Lite 19"
Aluminium
Companion Chair

Designed In The USA and Distributed By:

LIFESTYLE MOBILITY AIDS

Email : contactus@lifestylemobilityaids.com

Website : www.lifestylemobilityaids.com